

STM Healthcare

Overlays,
Positioning Products
& Accessories

because we **care**





STM Healthcare

Overlays, Positioning Products & Accessories

In addition to pressure care mattresses and cushions, STM Healthcare has been providing pressure reducing overlays, as well as positioning products, pillows and accessories to hospitals and the community for many years.

STM Healthcare overlays, pillows and accessories are designed to provide the hospital, nursing home and community nurse with a full range of first class quality products.

Whether a pressure care SnuggleDown overlay or a simple bed wedge you can be assured of STM's attention to detail and commitment to providing products that are, where applicable, fully supported by clinical evaluation.



Overlays	4
Cot sides	5
Floor mats	5
Charnley wedge support	6
Leg troughs	6
Bed wedge	6
Positioning products	7
Head supports	8
Pillows	9
Specifications	10 -11

Pressure Relieving Overlays

Foam and fibre overlays are designed to be used in hospitals, community or nursing homes by patients at risk of developing pressure sores.


Both products have been trialled over a continuous period at the Department of Medical Physics and Biomedical Engineering at Salisbury District Hospital.

SnuggleFoam Overlays



Lightweight multi-stretch, vapour-permeable cover fabric with zipped closure to one end to allow for interior inspection. The combustion modified high resilient foam (CMHR) interior has a profiled surface to aid the free flow of air and reduce the surface tension under high pressure points.

Snugglefoam overlays are also available with 100% visco-elastic foam (VE) or CMHR/VE layered foams.

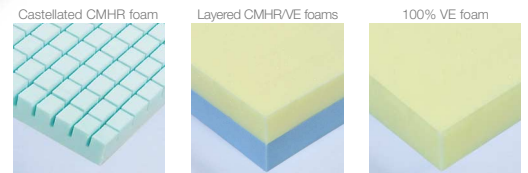

SnuggleDown Overlay

The luxurious SnuggleDown has deep fibre-filled rolls inserted across the width of the overlay, encapsulated in a fire retardant interliner. The carded hollow polyester filling offers superior levels of comfort and helps to prevent the onset of pressure ulcers.



Bed Accessories

For many patients who are at risk of developing pressure sores, their stay in hospitals or nursing homes can be made more comfortable with one or more of STM Healthcare's accessory products. The range offers a choice of overlays and bed accessories to compliment the mattress range, teaming pressure care with general comfort.

Padded Cot Sides / Rail Covers



Easy to attach, these combustion-modified high resilient foam cot sides envelop the bed's side rails and are secured by hook and loop attachments. With a waterproof nylon cover fabric, they are lightweight to handle and come in three standard sizes.

Floor Mat




Designed to cushion falls, the floor mat comes with a waterproof nylon, wipe-clean, removable cover and an anti-slip base cloth. With a combustion-modified foam interior, the mat is lightweight to handle and can be ordered in a folded version for under-bed storage.



Positioning & Bed Accessories

Leg Supports and Troughs

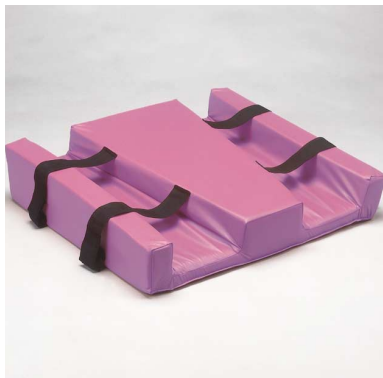

Stabilising the leg, the leg support raises the heel, reducing contact with the bed and aiding in the prevention of pressure sores. The double version is designed for use after hip operations and the legs are held by the troughed shaping and further secured by Velcro fixings. Combustion modified high resilient foams are combined with multi-stretch, vapour permeable, non-removable covers for enhanced comfort.

Charnley Wedge



An alternative to the double leg trough, for use after hip replacement operations, the wedge shape keeps the legs apart, while hook and loop straps prevent further movement. Combustion modified foam and multi-stretch, vapour permeable, non-removable covers for enhanced comfort.

Bed Wedge



The bed wedge is dual-positional allowing the patient to sit up or semi-recline. It has a removable, stretchable, vapour-permeable cover, which can be washed or simply wiped clean.



Positioning & Bed Accessories

Designed to aid the carer in the control of patient posture, positioning products are manufactured by Recticel Limited under licence from the Royal Hospital for Neuro-Disability and are intended for use with patients with neurological impairment.



Log Rolls

Stabilises patient, controlling posture whilst in the side lying position. Made from combustion modified high resilient (CMHR) foam with a multi-stretch, vapour permeable, removable cover.

Log rolls are also available with a terry towelling cover.



T Rolls

Similar to the Log Roll but the added structure controls spasticity. Constructed from CMHR foam with a multi-stretch, vapour permeable, removable cover.


T rolls are also available with a terry towelling cover.



E and F Rolls

Used in supine, the F-roll is designed for clients with windswept lower limbs (either left or right sided).





The E-roll, used in supine, helps to maintain symmetrical alignment of the lower limbs where there is a tendency for abduction, adduction or a combination of both.

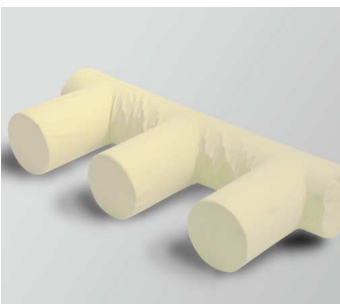

Positioning & Bed Accessories

Head Positioning Supports

These products feature a sculptured design for specific head positioning whilst undergoing treatment or recovery. Each support has a water resistant, polyurethane spray coating that can be wipe cleaned.

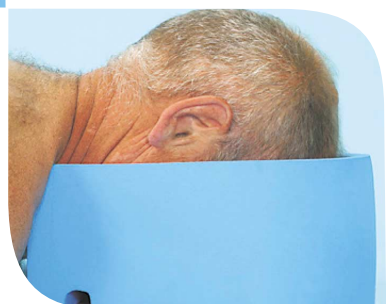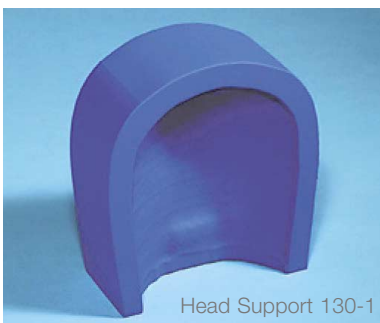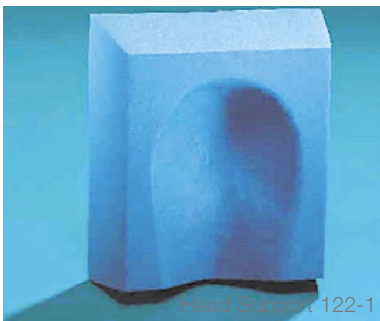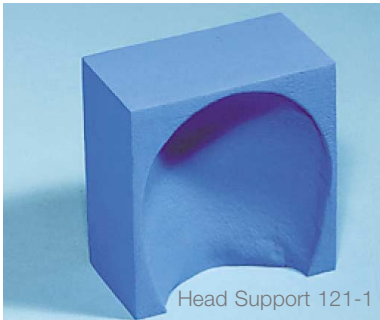

Foam & Fibre Pillows



Choose from the luxurious range of pillows developed by STM Healthcare to meet everyday requirements.

From cotton-covered standard pillows, shaped pillows and bolsters to pillows with top-of-the-range multi-stretch vapour permeable fabric covers, every STM product offers top quality and excellent value for money.

Foam interior

Manufactured with ultra soft SAFEGUARD® CMHR foams to ensure maximum comfort and product durability.



Fibre interior

Manufactured using hollow polyester fibre with excellent bulk to weight ratio to provide a light pillow with fullness.


Reduced fibre-to-fibre cohesion maintains high levels of comfort and loft.

Many hospitals use STM pillows for their replacement programmes.



Paramount Pillow

A shaped fibre-filled pillow that covers the upper body whilst resting. Hidden pockets provide extra support for the lumbar region, elbows and arms. Ideal for nursing mothers or when support is required whilst sitting up in bed.

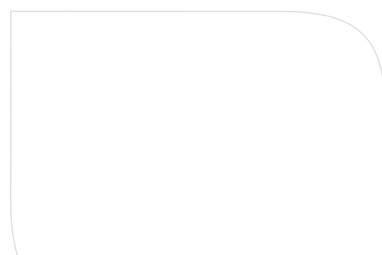

STM Healthcare

Overlays, Positioning Products & Accessories

		overlays				positioning and accessory products					
		SnuggleDown	SnuggleFoam			Cot sides rail covers pair	Safety mat flat	Safety mat folding	Double leg trough	Leg trough	Charnley wedge
			castellated CMHR	layered CMHR/VE	100% VE						
Ordering code		STM14 STM14D	STM20 STM20D	STM20LV STM20LVD	STM20V STM20VD	STM18S STM18M STM18L	STM35	STM35F	STM16D	STM16	STM27
Risk category	See note 1	low/medium	low/medium	low/medium	low/medium						
Construction											
	Safeguard® CMHR foam		●			●					
	Safeguard® CMHR & visco elastic foams			●							
	Visco elastic foam				●						
	Hollow polyester fibre	●									
	CME foam						●	●			●
	FR foam								●	●	
	FR foam PU coated										
	Fire retardancy, see note 2	●	●	●	●	●	●	●	●	●	●
Cover fabric											
	Multi stretch, wipe clean, waterproof, vapour permeable, PU coated nylon	●	●	●	●				●	●	●
	Removable washable FR cotton										
	Waterproof fabric					●	●	●			
	Sewn cover seams	●	●	●	●	●	●	●	●	●	●
	High frequency welded seams										
	Fixings								velcro		velcro
	Anti-slip base						●	●			
Colour		plum	plum	plum	plum	blue or green	blue	blue	plum	plum	peach
Standard sizes	cm	STM14: 198x89 STM14D: 198x137	single size: 200x90x7.5 double size: 200x137x7.5			STM18S: 107 STM18M: 140 STM18L: 152	200x66x7	200x66x7	48x41	48x63	41x34x15 43x36x15 30x36x15
Other sizes to special order		○	○	○	○	○	○	○	○	○	○
Warranty period	Months	12	12	12	12	12	12	12	12	12	12

● standard ○ optional

Note 1 Indication of risk - not a prescriptive criteria

Note 2 Foam overlays, fibre overlays, foam accessories and foam pillows BS 7177 : 2008 medium hazard; Fillings comply with: Furniture and Furnishings (Fire) (Safety) Regulations 1988

Fibre accessories and fibre pillows BS 7175 : 1989 medium hazard

positioning and accessory products

pillows

Log roll	T roll	E roll	F roll	Bed wedge	Head support 1	Head support 1B	Head support 2	Head support 3	Head support 4	Cotton fibre filled	Cotton foam filled	Water-proof fibre	Water-proof foam	V shaped	Para-mount
PUT2	PUT1	PUT10	PUT11	STM29	121-1	122-1	130-1	131-1	132-1	FIB/P3	LNF/P2	FIB/BF	LNF/P1	FIB/V2	FIB/P5

●	●	●	●	●												●
										●		●		●	●	
●	●	●	●	●	●	●	●	●	●	●	●	●	●	●	●	●
●	●	●	●	●								●	●			
●	●	●	●	●						●	●			●	●	
●	●	●	●	●						●	●			●	●	
												●	●			
cream	cream	cream	cream	peach	blue	blue	blue	white	white	white	white	grey/ivory	grey/ivory	white	white	
20x63	71x20x20	105x45x20	83x45x20	61x61x40	19x18x10	27x26x10	27x24x11	28x22x12	30x25x10							
○	○	○	○	○	○	○	○	○	○	○	○	○	○	○	○	○
12	12	12	12	12	12	12	12	12	12	12	12	12	12	12	12	12



STM Healthcare is part of The Recticel Group, Europe's leading manufacturer of polyurethane foams, with over 100 production sites in 16 countries throughout Western and Eastern Europe. The UK's foam manufacturing plant, based at Alfreton, Derbyshire, incorporates state-of-the-art, environmentally friendly technology.



Foam for Care is Recticel's European healthcare initiative, specialising in the manufacture and supply of general or bespoke medical devices. STM Healthcare is a founder member of Foam for Care, bringing together healthcare divisions from around the Recticel Group, developing innovative solutions for the healthcare market.



Latex

Suppliers to STM Healthcare of covering fabrics, sundry items and filling materials, confirm that their materials used in the manufacture of STM Healthcare products do not contain natural rubber latex.



We reserve the right to alter without prior notice any details contained herein in the interest of product development. Safeguard® is a registered trademark in the name of Recticel Limited. SnuggleDown and SnuggleFoam are trademarks in the name of Recticel Limited.



Member of British Healthcare Trades Association



STM Healthcare

Azalea Close, Clover Nook Industrial Park
Alfreton, Derbyshire DE55 4QX

Tel: 01773 830426
Fax: 01773 830427

stmhealthcare@recticel.com

www.stm-healthcare.co.uk



A DIVISION OF RECTICEL LIMITED